

## **How to Mount Side Rails on Existing Bed**

- Place side rail holder on side of bed.
- Check alignment of holes to prevent contact with screws in bottom of bed frame.
- Mark one hole and drill through both walls of frame tubing with a 5/16" drill.
- Attach side rail holder with enclosed 5/16" bolt assembly.
- Place side rail in holder and line up holder by leveling the top of side rail.
- Using side rail holder as a guide, drill another 5/16" hole making sure side rail remains level.
- Place 5/16" bolt through hole and tighten both bolts.
- Repeat on opposite side of bed.

Side rails have a very slight bend in them.

When placing them in the holders, be sure the outside of the bend is to the inside of the bed so the rail will slide properly.

**\*Be sure 5/16" bolts will not come in contact with screws through bottom of the bed frame**