


## Replacement of Upholstered Top:

- To remove an upholstery section top from the frame it is necessary to remove the screws from the bottom side of the frame that hold the upholstered top to the frame.
- Put the bed in a flat position, pull the upholstered section top from the frame and lay it over onto the bed. This will expose the bottom side of the upholstered top.
- Remove the staples holding the flap to the upholstered top and place the top aside.
- Put the new upholstered top in the same position as the previous one was on the bed and reattach the flap to the new top in approximately the same location as it was on the previous top.
- Line the new top up with the frame and using a 1/8" bit, pre-drill holes for the screws that hold the top to the frame.
- Attach the top to the frame with the screws removed earlier. Do not over tighten the screws –they can strip easily. The flap is the portion of vinyl upholstery in between the Imaging Drop Section and the bed.
- Repeat the process for each section on your exam bed.

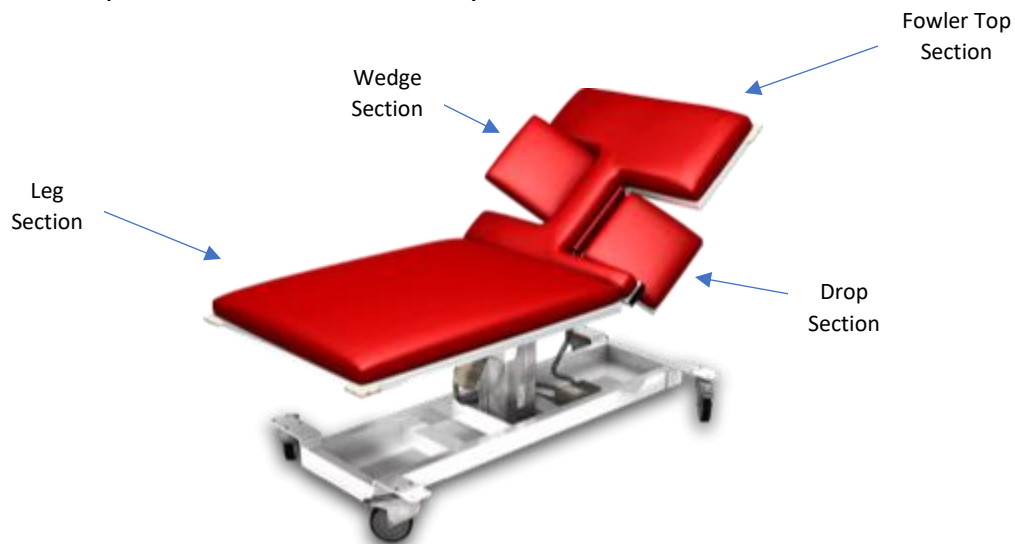

If you have any questions about replacing the upholstery, please call Heritage Medical Products, Inc. at 888.256.8748 or email [sonobed.info@gmail.com](mailto:sonobed.info@gmail.com)